



## Mirage Quattro™

### FULL FACE MASK FITTING INSTRUCTIONS (Size selection)

The Mirage Quattro Full Face Mask fitting template is designed to help determine the correct mask size (**X-Small, Small, Medium or Large**) using the height from the chin crease to the bridge of the nose.

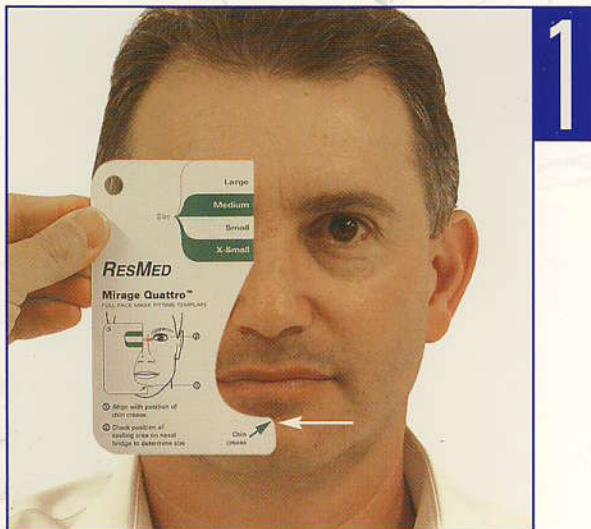

Align the fitting template to the chin crease.

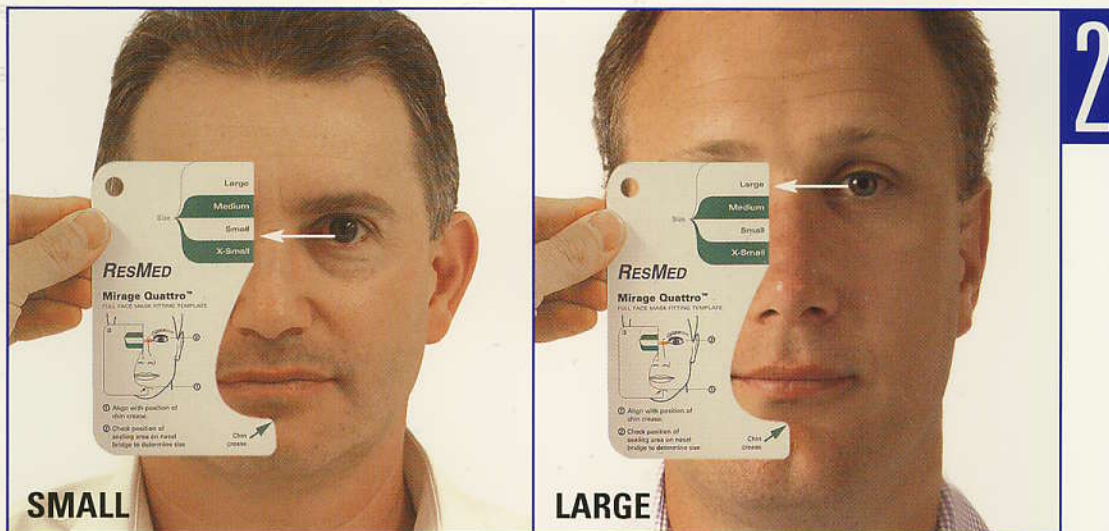

Check position of sealing area on nasal bridge to determine size.

Size

Large

Medium


Small

X-Small

**RESMED**

**Mirage Quattro™**

FULL FACE MASK FITTING TEMPLATE

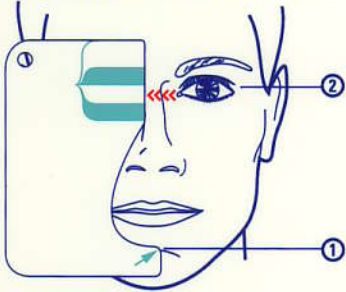

- ① Align with position of chin crease
- ② Check position of sealing area on nasal bridge to determine size

Chin  
crease

